


# Os Finder™

## Cervical Os and Canal Dilator

The Os Finder naturally seeks and gently opens the cervical Os. Made of an extremely smooth and malleable Teflon-like material, this instrument is ideal for use in intracervical dilation or uterine sounding, or for intrauterine procedures.

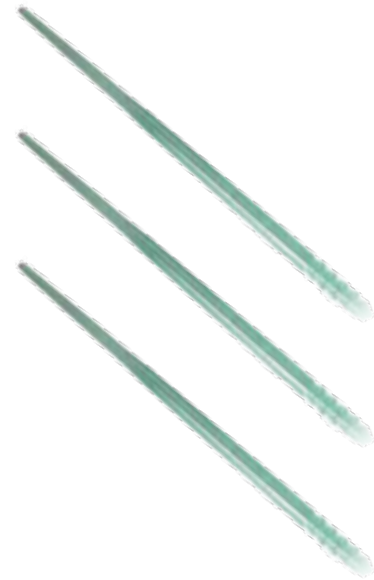

### DEVICE DESCRIPTION

The sterile, single use Os Finder™ naturally seeks and gently opens the cervical Os. It is made of extremely smooth and malleable teflon-like material. The dilator is 21.5 cm long with a taper from 2 to 4 mm. CAUTION: Federal restricts this device to sale by or on the order of physician.

### CONTRAINDICATIONS

This instrument should not be used under any of the following conditions:

- Pregnancy or suspicion of pregnancy
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum
- In the case of cervicitis, endometritis, tubo-ovarian inflammatory disease or pelvic inflammatory disease
- Invasive cancer that is visible on examination

### PRECAUTIONS

Do not use the Os Finder if the package or instrument are damaged.

### INDICATIONS FOR USE

This instrument is ideal for use in intracervical dilation, uterine sounding, or for intrauterine procedures.

### DIRECTIONS FOR USE

The Os Finder is used to find the external Os by simply pushing the narrow end gently against the cervix until it finds its way. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal Os. Dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

Item #	Description
1176	Os Finder (50/Box)

